



# GRIP-LOK® APPLICATION GUIDE

## 3300M Medium for Securement of Surgical Jejunostomy Device


Prepare the skin according to the standard hospital protocol for dressing application.

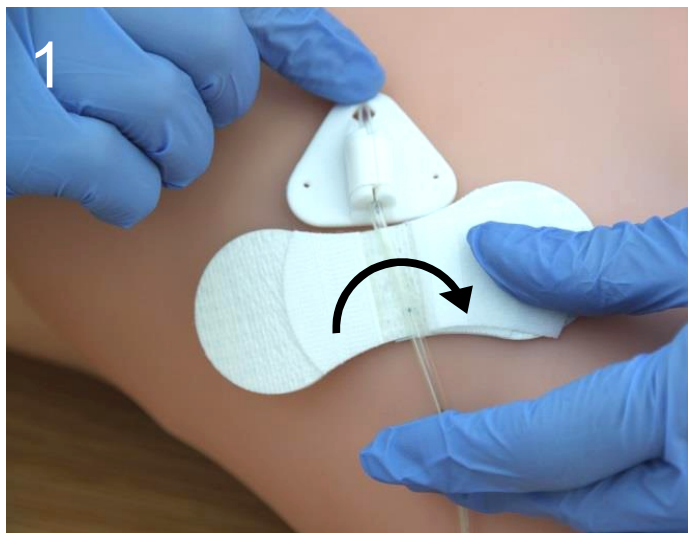


Open the top flap and remove the inside liner. Slide the Grip-Lok® under the jejunostomy tube.

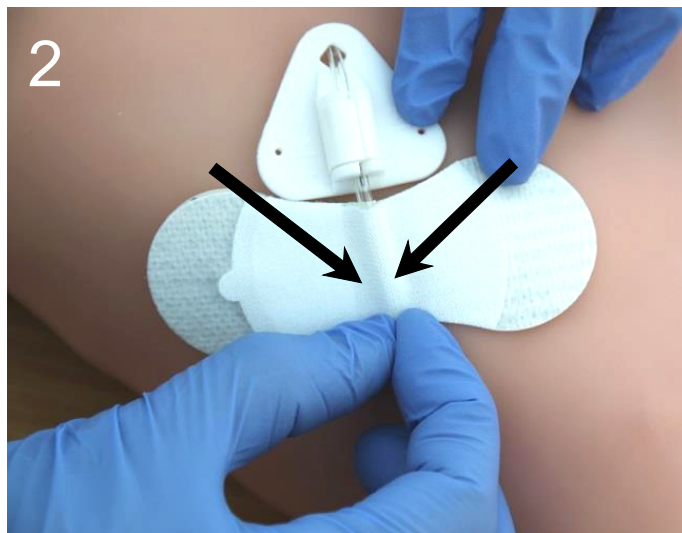

Firmly secure the jejunostomy tube by closing the top flap and then **pinching around and along the length of the tube.**

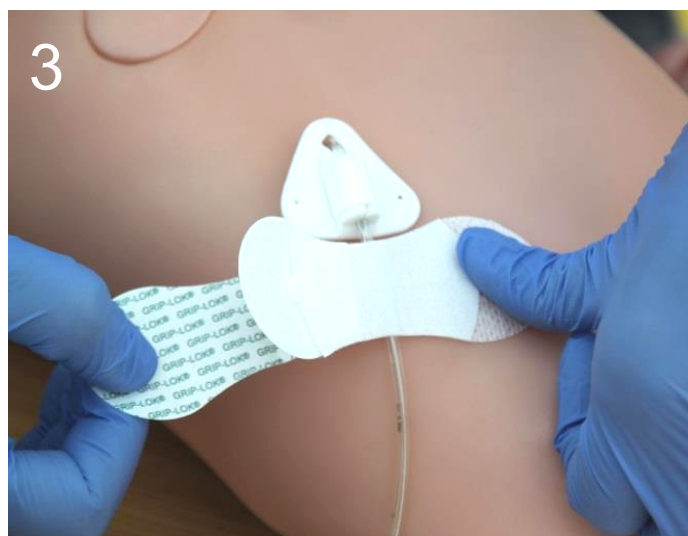

Hold the Grip-Lok® and tube in position by placing thumb on the tabbed side, away from the centre section. Pull the lower release liner on the non-tabbed side and adhere to the skin.

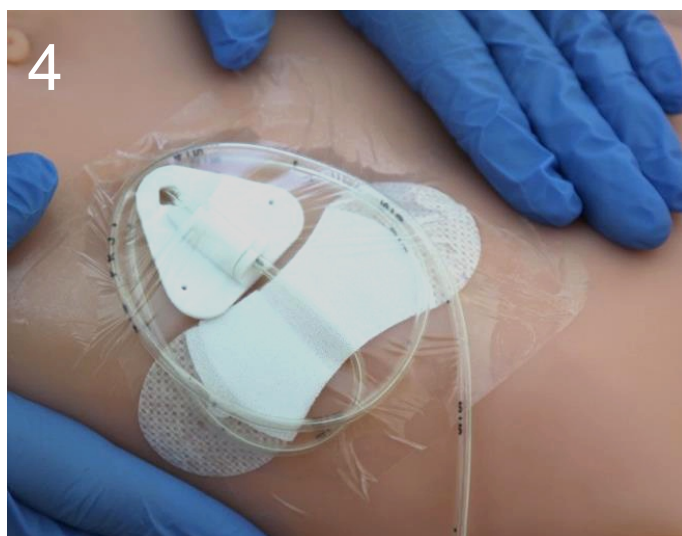

Holding the Grip-Lok® on the tab side, remove the remaining release liner and secure to the skin. Then gently press down around the edges and top surface of the Grip-Lok® to ensure the adhesive connects well to the skin and to the tube. **OPTIONAL: FILM DRESSING (TEGADERM) CAN BE APPLIED.**



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



info@blueboxmedical.co.uk

Grip-Lok® NHS Supply  
Chain code: ELW704  
Available on prescription  
(FP10) code: 3300M-10PK



BLUE BOX  
MEDICAL LTD

www.blueboxmedical.co.uk



AG002  
01/17 - D