



# GRIP-LOK® APPLICATION GUIDE

## 3300MWA Medium Wide Adhesive for PICC/Midline Securement


Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

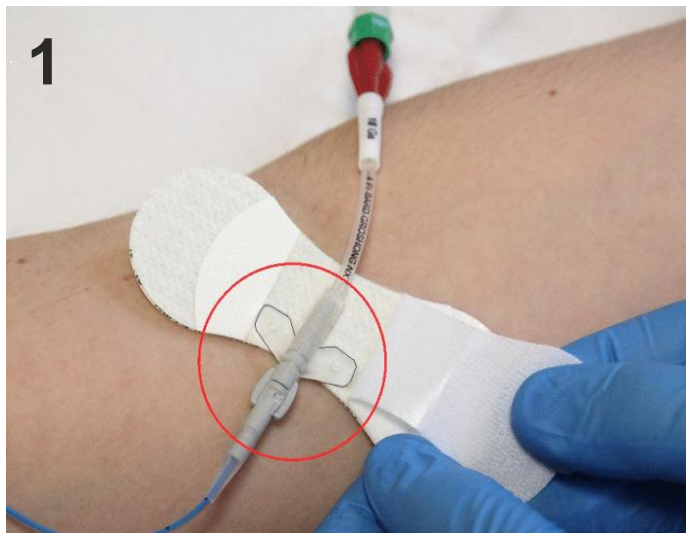


Open the top flap and remove the inside liner. Slide the Grip-Lok® under the catheter and ensure the winglets/hub of the PICC line are positioned to the top of the adhesive section. **This is very important.**

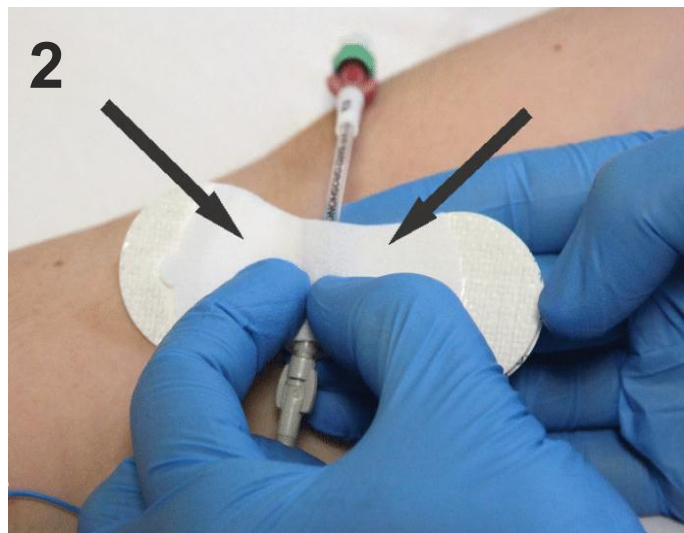

Firmly secure the PICC line by closing the top flap and then **pinching around the winglets/hub and along the length of the catheter.**

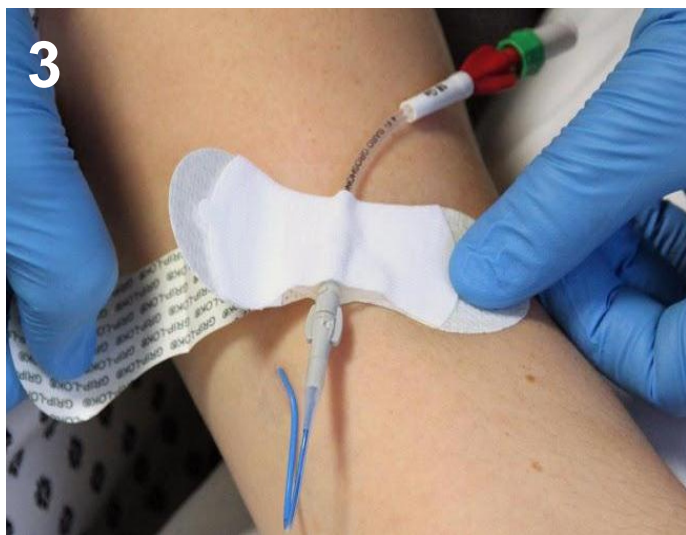

Hold the Grip-Lok® and catheter in position by placing thumb on the non-tab side, away from the centre section. Pull the lower release liner on the tabbed side and adhere to the skin.

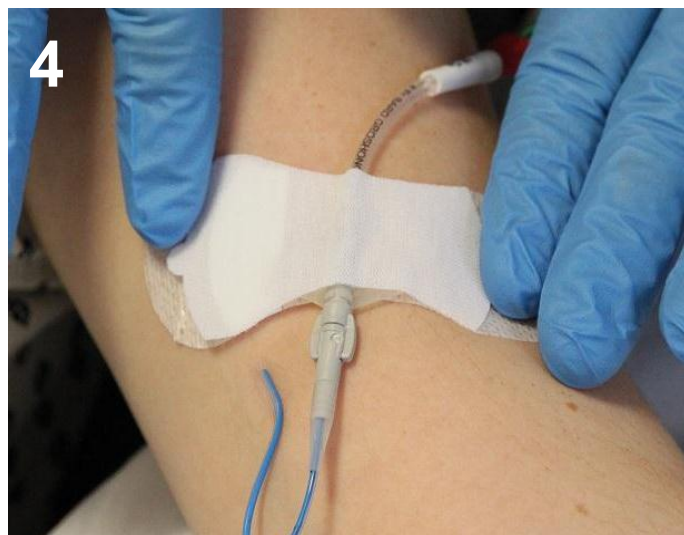

Holding the Grip-Lok® on the tab side, remove the remaining release liner and secure to the skin. Then gently press down around the edges and top surface of the Grip-Lok® to ensure the adhesive connects well to the skin and to the PICC.



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



Grip-Lok® NHS Supply  
Chain code: ELW703

info@blueboxmedical.co.uk



BLUE BOX  
MEDICAL LTD

www.blueboxmedical.co.uk



AG003  
10/15 - D