


# GRIP-LOK<sup>®</sup> APPLICATION GUIDE

## 3400LFC for Foley Catheter Securement

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.




1 Open the Velcro<sup>®</sup> lid.

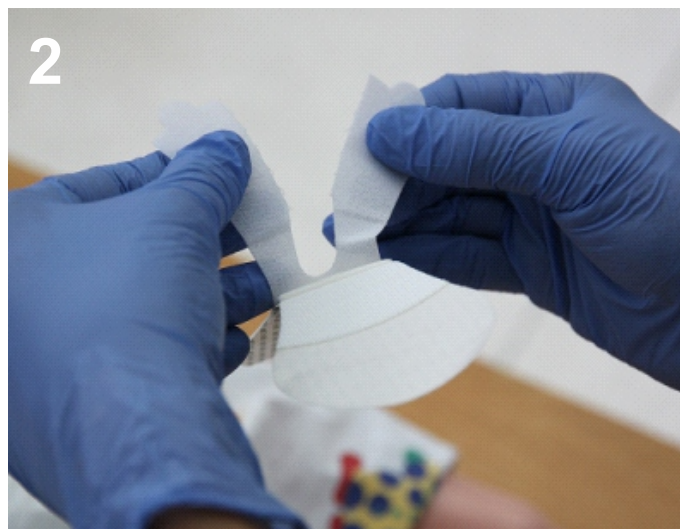

2 Split the lid along the perforations.



3 Place the bifurcation crotch centrally on the Grip-Lok<sup>®</sup>.

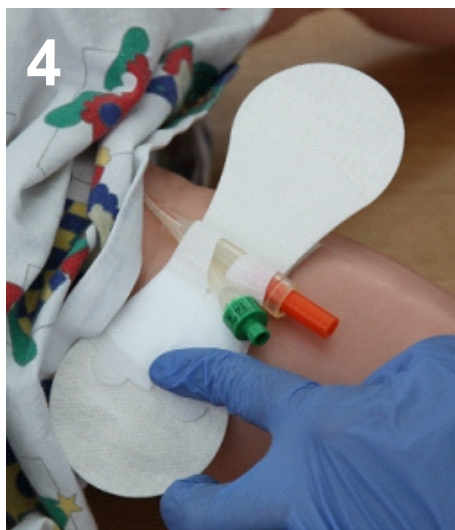

4 Place half the Velcro<sup>®</sup> lid over the top of the catheter and secure down onto the Velcro<sup>®</sup>. Then place the remaining half in between the bifurcation crotch and secure down onto the Velcro<sup>®</sup>.



5 Pull the paper backing from one side of the Grip-Lok<sup>®</sup>. Then pull the paper backing from the other, to secure the Grip-Lok<sup>®</sup> in the desired position on the skin.



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



info@blueboxmedical.co.uk

Grip-Lok<sup>®</sup> NHS Supply  
Chain code: ELW 368



BLUE BOX  
MEDICAL LTD

www.blueboxmedical.co.uk



AG004  
01/17 - C