



# GRIP-LOK<sup>®</sup> APPLICATION GUIDE

## 3304MCS-BA for Bard PICC Securement

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.

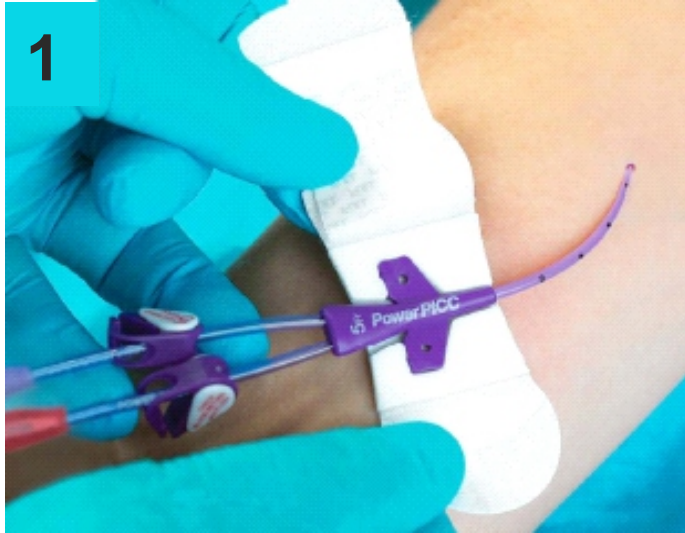

Open the top flap and slide the Grip-Lok<sup>®</sup> under the catheter hub. Position the catheter hub down into the foam cut-out and press gently to the adhesive.


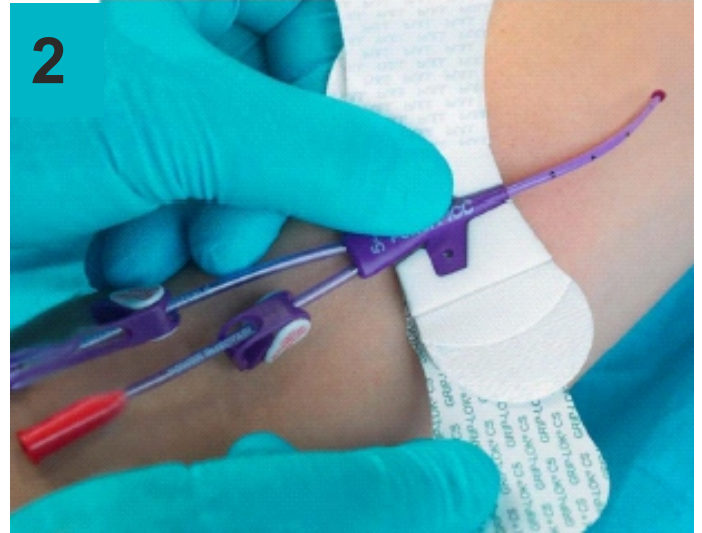


Pull and remove one side of the bottom release liner while holding the Grip-Lok<sup>®</sup> in position. Then pull and remove the other side of the bottom release liner.



Pull and remove the inner release liner under the top flap to expose the adhesive.

*Alternative technique: Secure the Grip-Lok<sup>®</sup> to the catheter hub before adhering to the skin.*

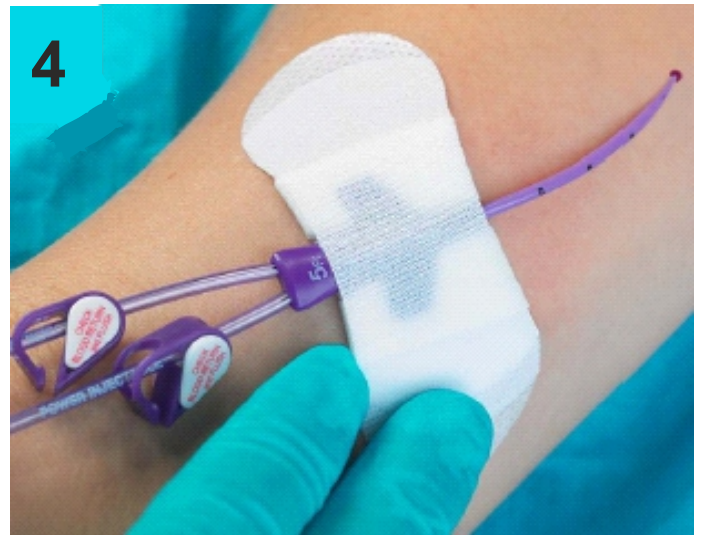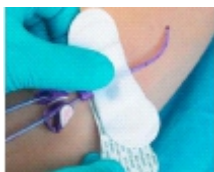

Secure the top strap over the catheter hub and press the adhesive in place. The top strap can be peeled back to inspect and adjust the catheter hub.



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



Grip-Lok<sup>®</sup> NHS Supply  
Chain code: ELW 365

info@blueboxmedical.co.uk



BLUE BOX  
MEDICAL LTD

www.blueboxmedical.co.uk



AG005  
01/17 - C