


GRIP-LOK® APPLICATION GUIDE

3601CVC for CVC Securement

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



1 Open the top fabric over-strap and remove the inside release liner.

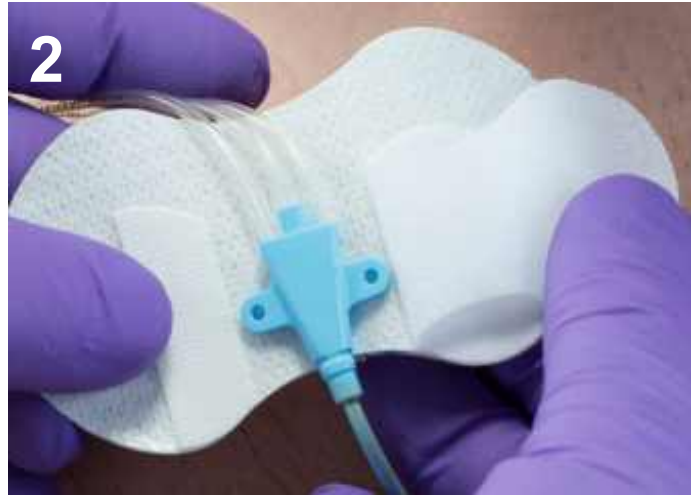

2 Slide the Grip-Lok® under the catheter centering the hub in the exposed base adhesive area.



3 Secure the fabric over-strap down over the catheter and press around and on top of the catheter and lumens to firmly attach the adhesive.



4 While holding the Grip-Lok® and catheter in place with one hand, remove the bottom release liner from one side and adhere to skin. Then remove the other bottom release liner and secure that side to skin.



Grip-Lok® 3601CVC can secure on the chest or neck



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



Grip-Lok® NHS Supply
Chain code: ELW583

info@blueboxmedical.co.uk



BLUE BOX
MEDICAL LTD

www.blueboxmedical.co.uk



AG007
05/15 - C