

GRIP-LOK[®] APPLICATION GUIDE

3300MWA Medium Wide Adhesive for A-Line Securement

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



Clean hands using soap and water followed by alcohol gel. Using ANTT principles, prepare and cleanse the skin according to the standard operating procedure for A-Line insertion.



Open the top flap and remove the inside liner. Slide the Grip-Lok[®] under the A-Line. Ensure that the winglets of the A-Line are positioned so the access port can be seen.



Close the top flap and firmly secure the A-Line by pinching around the winglets and along the length.



While holding the Grip-Lok[®] in place with one hand, remove the two bottom release liners. Ensure the Grip-Lok[®] is firmly stuck down. A Tegaderm or IV dressing can also be placed over the Grip-Lok[®].



PLEASE NOTE - The A-Line should be placed so access to the port can be gained from the edge of the Grip-Lok[®].



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



Grip-Lok[®] NHS Supply
Chain code: ELW703

info@blueboxmedical.co.uk



BLUE BOX
MEDICAL LTD

www.blueboxmedical.co.uk



AG008
01/17 - D