

GRIP-LOK® APPLICATION GUIDE

3301MCS-BD for BD Arterial Cannula with FloSwitch® Securement

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



Open the top flap, remove the interior liner and hold the flap open.



Slide the Grip-Lok® under the arterial cannula and adhere in the exposed adhesive cut-out space provided.



Hold the Grip-Lok® in position while removing the paper backing from one side, then the other, to secure on the skin surface.

Note: For added skin protection, skin prep may be used prior to application of the securement device. Replace the securement device if soiled or saturated with fluid. Use of alcohol swab will ease removal of the securement device from the skin.



Secure the top flap over the arterial cannula with the FloSwitch® portion coming through the centre cut-out.



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



info@blueboxmedical.co.uk

Grip-Lok® NHS Supply
Chain code: ELW 591



BLUE BOX
MEDICAL LTD

www.blueboxmedical.co.uk



AG010
01/17 - C