


GRIP-LOK® APPLICATION GUIDE

3300MEPI for Epidural Catheter Securement


Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



Slide the Grip-Lok® under the catheter at the notch. Pull and remove one side of the bottom release liner while holding the Grip-Lok in position. Then pull and remove the other side of the bottom release liner.

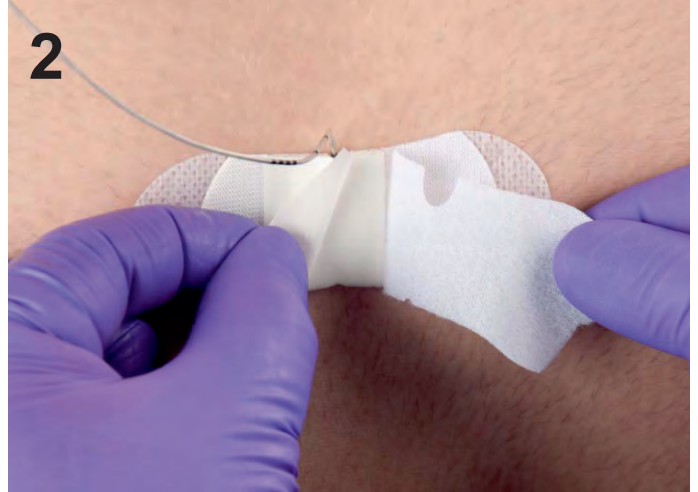

Open the top flap and remove the release liner.

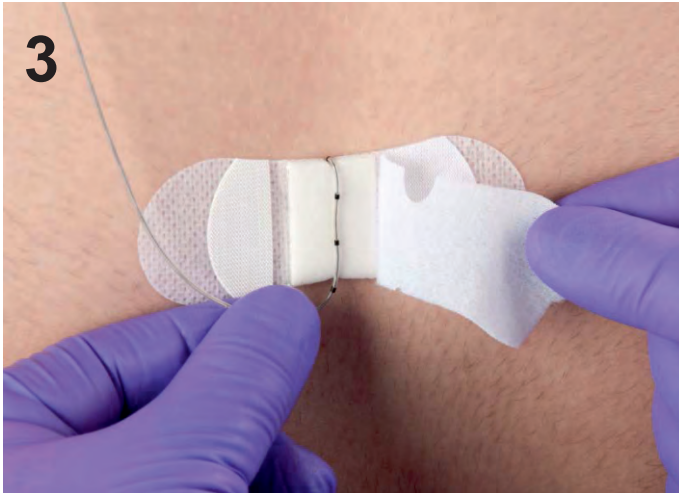

Place the catheter across the Grip-Lok® from notch to notch and gently press the catheter onto the upper adhesive.



Secure the top flap over the catheter and apply gentle pressure to the centre fabric area over the catheter adhesive.



A tape strip is provided to j-loop the catheter under the transparent dressing.

Note: to provide visual assurance of catheter position, you can make a note of the exposed catheter length markings at the point of exit from under the top flap or by marking on the top side of the 3300MEPI securement device.



Grip-Lok is a trademark of TIDI Products, LLC



Grip-Lok® direct order
code: 3300MEPI

Tel: 023 8066 9000

info@blueboxmedical.co.uk



BLUE BOX
MEDICAL LTD

www.blueboxmedical.co.uk



AG013
01/17 - C