

GRIP-LOK® APPLICATION GUIDE

2800FCS for Foley Catheter Securement

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion. Skin must be **CLEAN** and **DRY** prior to Grip-Lok® application.




Open the top fabric over-strap and separate it into two sections.



Place one section of the fabric over-strap over the drainage lumen and under the inflation lumen. Place the other fabric over-strap over the catheter.



Turn over the Grip-Lok® and pull the long release liner partly from the skin contact adhesive.

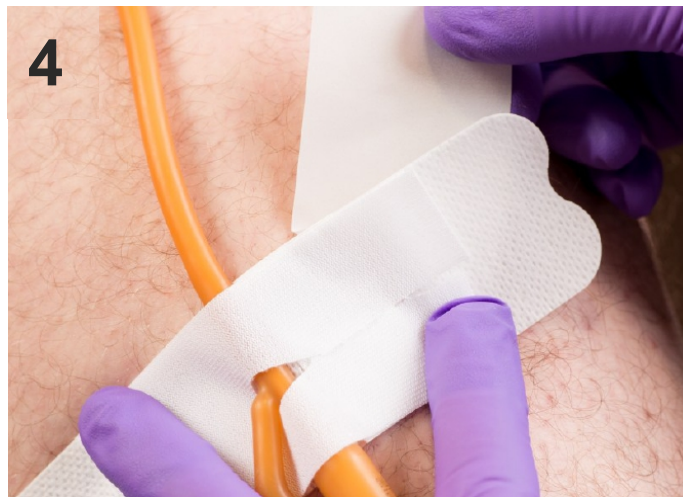

Pull the release liner from the long side of the Grip-Lok® and secure to the skin, then pull the release liner from the short side and secure to the skin.



Grip-Lok® 2800FCS will also fit triple lumen catheters.



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



Grip-Lok® NHS Supply
Chain code: FUB002
(also available on FP10)

info@blueboxmedical.co.uk



BLUE BOX
MEDICAL LTD

www.blueboxmedical.co.uk



AG014
10/16 -D