


GRIP-LOK[®] APPLICATION GUIDE

3500DFN for Dialysis Fistula Needle Securement


Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



1 Remove the two large adhesive release liners from one side leaving the centre strip and place the Grip-Lok[®] on the skin at the desired site. Repeat for the top side leaving the centre liner strip .

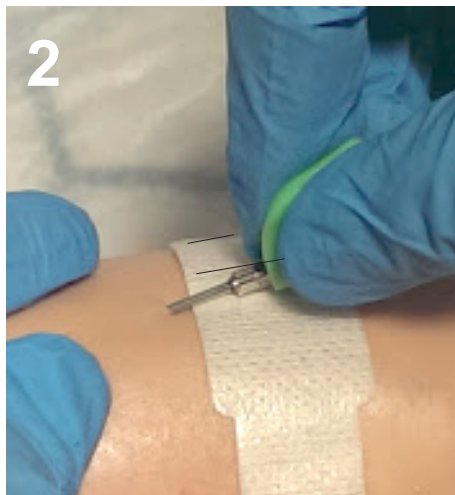


2 Insert the fistula needle so that it lays on top of the centre liner strip of the Grip-Lok[®] securement.

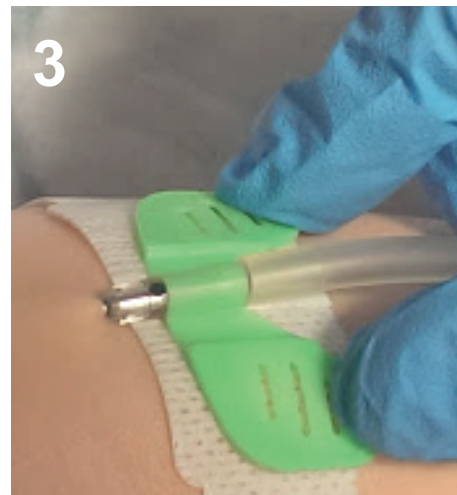


3 Press the needle's wings onto the adhesive area and begin the blood flow.



4 Fold over and press the adhesive around the wings.

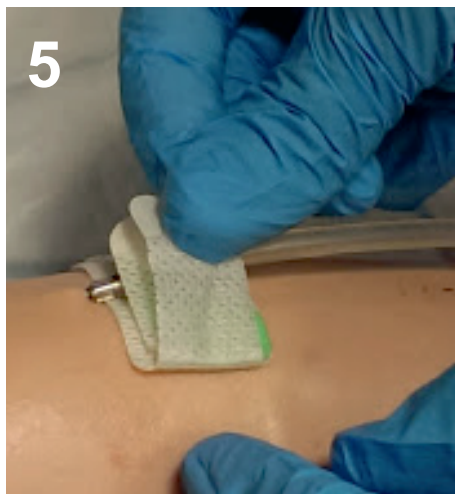

5 One side at a time, peel back the adhesive securement until both halves are released from the skin, leaving only the centre strip in contact with the skin.

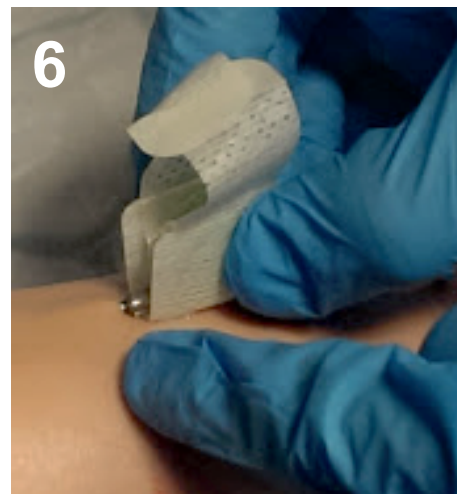

6 Withdraw the needle and dispose safely.



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



info@blueboxmedical.co.uk

Grip-Lok[®] direct order
code: 3500DFN



BLUE BOX
MEDICAL LTD

www.blueboxmedical.co.uk



AG015
01/17-C