

GRIP-LOK® APPLICATION GUIDE

3300M for IV Line Securement

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



Pull and remove one side of the bottom release liner and position the Grip-Lok® under the IV line. Then remove the other side of the bottom release liner.



Open the top fabric over-strap and remove the inner release liner to expose the adhesive.



Place the line/hub in the centre of the bottom adhesive strip.



Secure the top strap over the line/hub and press the adhesive in place. The top strap can be peeled back to inspect and adjust the catheter line/hub.



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



info@blueboxmedical.co.uk

Grip-Lok® NHS Supply
Chain code: ELW704
Available on prescription
(FP10) code: 3300M-10PK



BLUE BOX
MEDICAL LTD

www.blueboxmedical.co.uk



AG021
01/17 - D