


GRIP-LOK[®] APPLICATION GUIDE

3400L Large for Foley Catheter Securement

Prepare the skin according to the standard hospital protocol for dressing application. Skin prep or hair removal may be required on some patients for better adhesion.



Pull the paper backing from one side of the Grip-Lok[®]. Then pull the paper backing from the other side to secure the Grip-Lok[®] in the desired position on the skin.

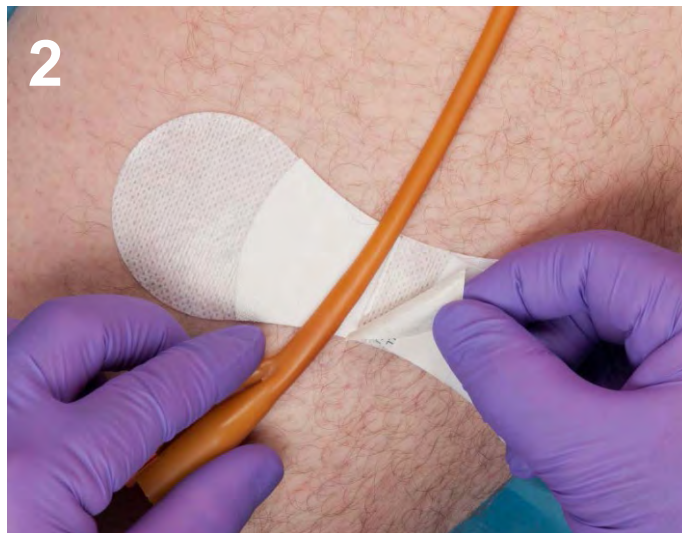

Open the top fabric over-strap and remove the paper liner to expose the adhesive strips.



Place the catheter centrally on the bottom adhesive area.



Secure the fabric over-strap over the catheter so that it is placed between the adhesive areas, then close the top over-strap.



Grip-Lok is a trademark of TIDI Products, LLC

Tel: 023 8066 9000



info@blueboxmedical.co.uk

Grip-Lok[®] NHS Supply
Chain code: FUB000
(also available on FP10)



BLUE BOX
MEDICAL LTD

www.blueboxmedical.co.uk



AG022
10/16 - C