


CORNEA-CARE™ APPLICATION GUIDE

Eye Protection Dressing

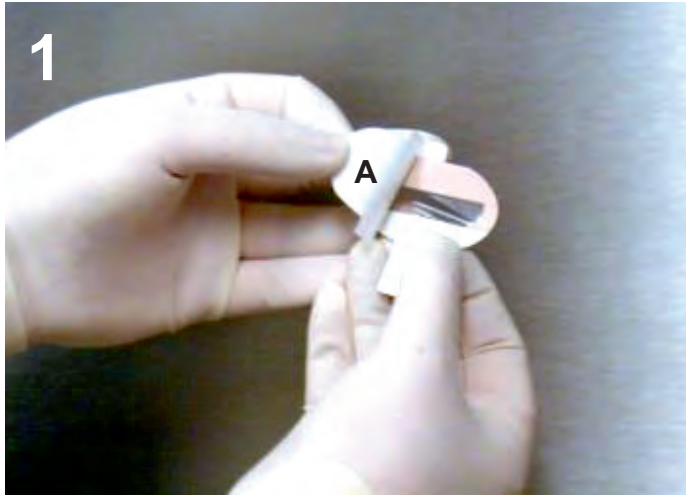

1 Remove the upper release paper (A) and hold the device between thumb and index finger.



2 Gently lift eyebrow with an index finger and place the upper adhesive portion (B) over the upper eyelid, below the eyebrow



3 Peel lower release paper and pull tab (C) to apply lower adhesive limb portion over the maxillary area of face.



4 Gently smooth the Cornea-Care to obtain eye closure.



**Direct order code:
CCA-01(500)**

At the conclusion of the procedure simply pull downward from the upper part.