


ACAPSIL[®] APPLICATION GUIDE


Please refer to the Acapsil Instructions for Use for comprehensive information on application, use and precautions.




INITIAL APPLICATION

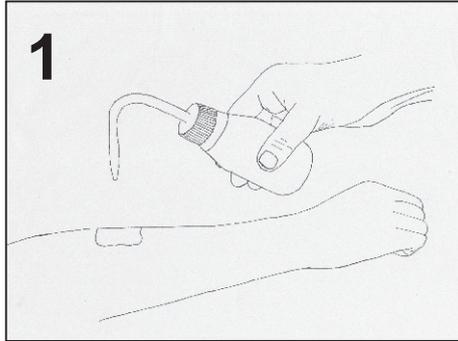


Clean the wound as thoroughly as possible using saline or tap water. Use a lint-free swab to help remove any undesired matter. **Do not** use Chlorhexidine.

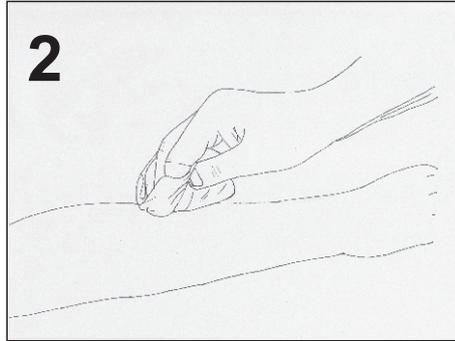

Dry the wound gently by dabbing it with a dry, lint-free swab.

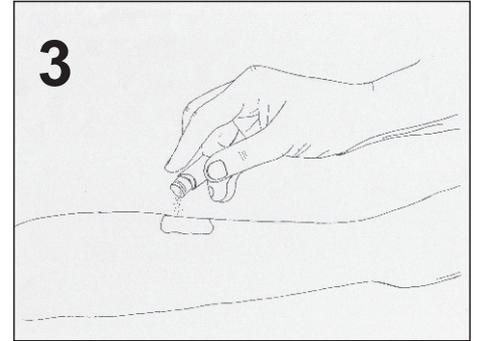

Sprinkle Acapsil directly onto the entire wound surface in an even layer of 1-3mm. Ensure that the entire wound surface as well as the wound edges are well covered.

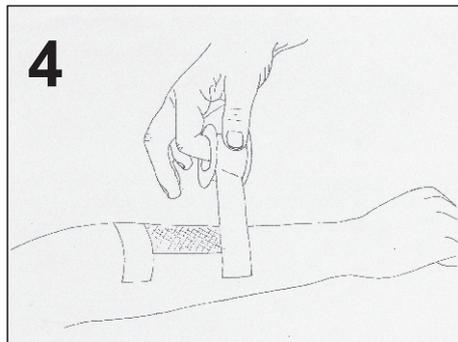

Cover with a light, permeable secondary dressing. This dressing must be dry and highly permeable to allow for evaporation of wound exudate and for the wound to breathe. Examples could be a 4-ply lint-free swab or a paper thin, dry, open mesh, highly permeable, non-impregnated contact layer dressing. Tape the dressing in place, ensuring the tape does not pass over the wound itself. **Do not** use impregnated dressings.

DRESSING CHANGE

- 5 Remove the secondary dressing slowly and carefully, without jerking.
- 6 Clean the wound while taking care not to disturb or disrupt the newly formed tissue in the wound bed.
- 7 Repeat steps 2 - 4.
- 8 Repeat steps 5 - 7 once daily until the wound is clean, ie free from slough, pus, maceration and biofilm. If the secondary dressing is soaked within the first 12 hours after application of Acapsil, repeat steps 5 - 7.
- 9 Once the wound is clean, discontinue the use of Acapsil and simply cover with an atraumatic, dry, open mesh, fully permeable, non-impregnated contact layer dressing, allowing the wound surface to breathe. This can be covered with a 4-ply gauze as this combination will still allow the wound to breathe.

Acapsil must always be stored in a refrigerator (2° - 10°)

